

# Handmade Face Masks

Reprinted from: <https://www.craftpassion.com/face-mask-sewing-pattern/>, with insight from Covenant HealthCare COVID-19 Infection Prevention Team.

Here is a video link to watch as a tutorial: <https://youtu.be/fT7R1jLvTDs>

## Material Needed:

Main fabric (100% cotton): 12”x6”, prewashed

Lining fabric (100% cotton): 10”x6”, prewashed

Round Cord Elastic: 16”

Face Mask Sewing Pattern without seam allowance **OR** Face Mask Sewing Pattern with seam allowance (located at end of pattern)




## Instructions:





1. Print out the Face Mask Sewing Pattern – Do not scale the printing.
2. Pin the templates onto the fabric, trace and cut out the fabric pieces.
3. Sew, clip curve along on the seam allowance, turn face mask right-side out. Topstitch.
4. Sew ears to the face mask. Insert elastic cord to the ear-pocket





\*If you have a Cricut Machine or are a Silhouette User, please see website for settings for cutting. (<https://www.craftpassion.com/face-mask-sewing-pattern/>)

## More Details:

Do not use the pattern directly from the website, it is not scaled to the correct size. Use the pattern attached to this document.

	<p>Fold the main fabric into halve with the wrong side facing each other, pin the paper pattern onto the double-layered fabric. Cut the fabric with 1/4" allowance, except the ear side. Cut the fabric at the ear side with 1" seam allowance.</p> <p>Insert the tracing paper between the layer, trace sewing lines with wheel.</p> <p>Remove pins and paper pattern, get set to sew.</p>
	<p>Repeat the same as the main fabric, except the ear side seam allowance. Cut the fabric at the ear side with zero seam allowance.</p>
	<p>Turn the face mask main piece with right side facing each other, sew the centerline of the facemask.</p>

		<p>Clip curve along with the seam allowance, be careful and don't accidentally clip on the stitches.</p>
		<p>Press seam allowance to one side, topstitch near to the seam line on the side where the seam allowance is.</p>
		<p>Repeat the sewing on the lining part.</p>
		<p>Flip the lining and the place onto the main face mask, align the center seam line, pin and sew the top and bottom lines.</p>

	<p>Turn the face mask right side out. Press seam neatly and topstitch near the edge.</p>
	<p>This is how it looks like from the inside at this stage. The main fabric of the side ear area is longer than the lining piece. This is going to be the pocket of the elastic band.</p>
	<p>Fold in 1/4" at the raw edge of the side ear, fold again so that the first fold edge is 1/4" from the lining raw edge.</p> <p>Sew a straight line down, don't forget to perform a few back stitches to lock the thread end.</p>
	<p>Measure and cut the length of the elastic band needed (usually around 6"-8" depending on the size of the face).</p> <p>Insert the elastic band through the pocket and tie a dead knot to the ends.</p>





Repeat the same to the other side. Pull the elastic band until the knots are hidden in the pocket.

